


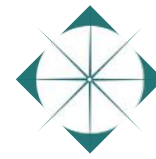

BodiTrak Balance System




BODITRAK BALANCE SOFTWARE INSTALLATION FOR IPAD



1

LOCATE BODITRAK BALANCE APP


Connect your iPad to the internet. Navigate to the Apple Store, tap on the Search icon. Type “**Boditrak Balance**”, locate the BodiTrak Balance application as shown on the picture below and tap on it.

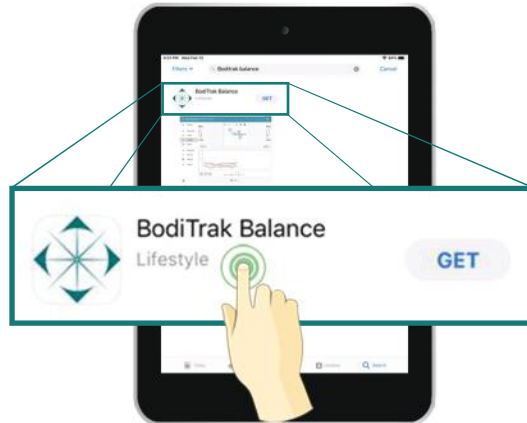

2

DOWNLOAD BODITRAK BALANCE APP

Once on the BodiTrak Balance screen of the Apple Store tap on **GET** to download the application using your Apple account credentials.

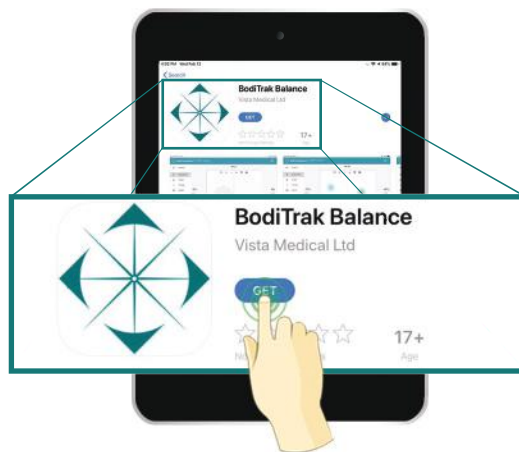

You can now connect to your BodiTrak2 Sensor mat through your iPad's Wi-Fi, and start using the BodiTrak Balance application.

Note: If you run into any issues during the installation process, feel free to contact our technical support team: **1-800-847-3157, techsupport@vista-medical.com**



Vista Medical, LTD.
A Patient Tech Company
Winnipeg, MB Canada
800-847-3157
www.boditrak.com
info@vista-medical.com



For more information about
BodiTrak Systems:
800-822-3553