



BodiTrak Balance System




BODITRAK BALANCE SOFTWARE INSTALLATION FOR MAC



1 GO TO SYSTEM PREFERENCES


Click on the Apple Icon and select "System Preferences".

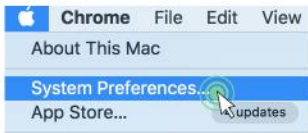

2 SECURITY AND PRIVACY




3 CLICK THE LOCK TO MAKE CHANGES

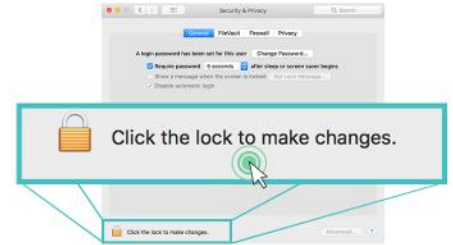


4 INPUT APPLE ID AND PASSWORD

Enter you User Name and Password to unlock Security and Privacy preferences.

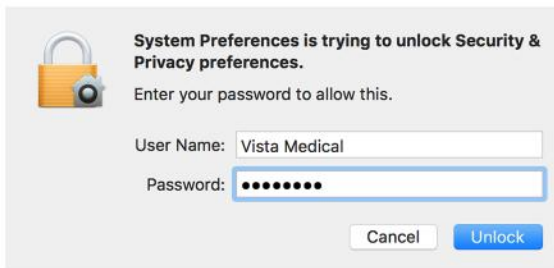


5 ALLOW THE DOWNLOAD

Make sure the option App Store and identified developers is selected.

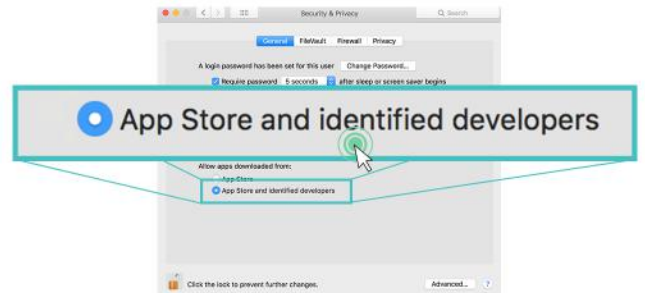

6 BODITRAK BALANCE SOFTWARE DOWNLOAD LINK

To download the BodiTrak Balance Software, please go to: [BodiTrak Balance Software Link](#)
Click "Next".

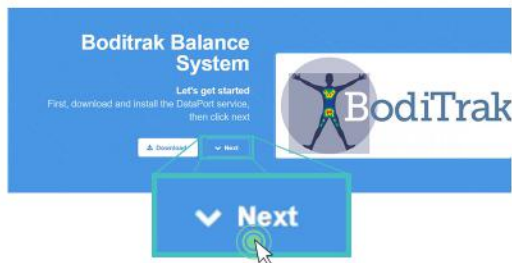

7 DOWNLOAD BODITRAK BALANCE SOFTWARE

Click on **Download** on the website to get the software. Wait until it is completed.




8 OPEN FINDER

Click on the downloaded file and select **Show in Finder**.



9 LOCATE BODITRAK BALANCE APP

Once in the Finder, locate the BodiTrak Balance app.

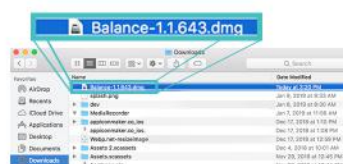

10 OPEN THE APP


Control-click the app icon, then choose **Open**.



11

CONFIRM TO OPEN

A new window will come up asking if you are sure you want to open, Click **"Open"**.

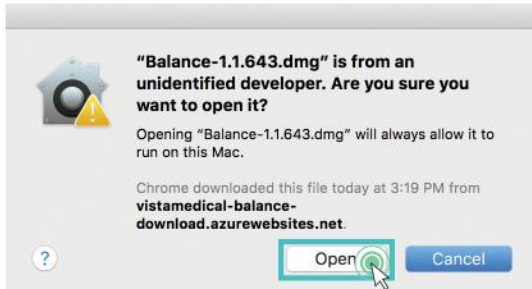

12

DRAG TO INSTALL

Drag the Balance logo to the Applications folder to install the BodiTrak Balance App.



13

CLICK ON FINDER


Click on the **Finder** icon located at the bottom of the screen.



14

RUN BODITRAK BALANCE APP


Go to applications, locate the BodiTrak Balance Application and double click on it.

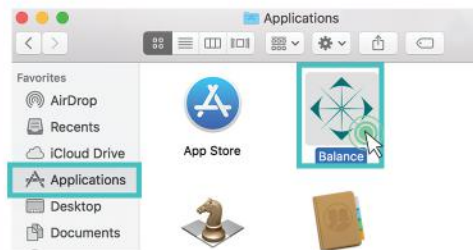

15

CLICK OPEN

Click on **"Open"**.

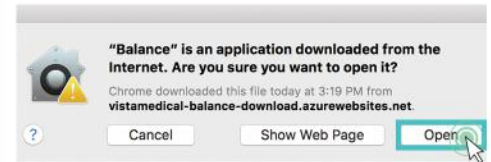

BodiTrak Balance App installation is completed at this point. You can now connect the Pressure Sensing Mat to the computer and start using the software.

Note: If you run into any issues during the installation process, feel free to contact our technical support team: **1-800-847-3157, techsupport@vista-medical.com**



Vista Medical, LTD.
 A Patient Tech Company
 Winnipeg, MB Canada
 800-847-3157
 www.boditrak.com
 info@vista-medical.com



For more information about
 BodiTrak Systems:
 800-822-3553