


How to Export All Data to Excel

➤ Exporting All Patients Data to an Excel Report

- On any of the three available platforms (Windows/Mac/iOS)
 1. Click/Tap on the settings icon ⚙
 2. Click/Tap on “Export Report”
 3. Click/Tap on “OK”

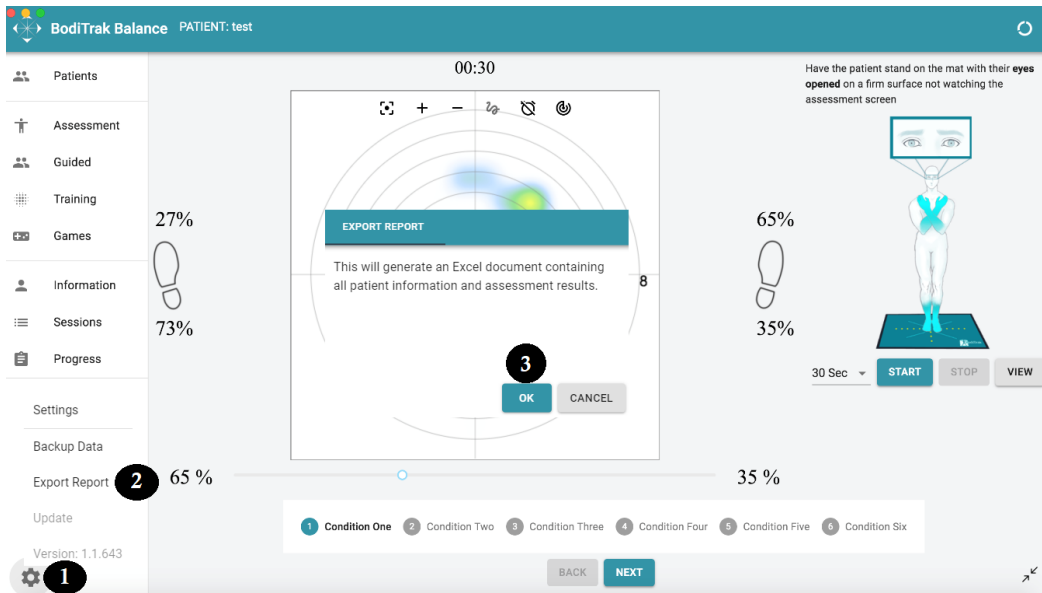


- After clicking “OK” on Windows
 1. Select location to save PDF file to
 2. Give the PDF file a name
 3. Click on “Save”




BodiTrak Balance System

Guided Assessment

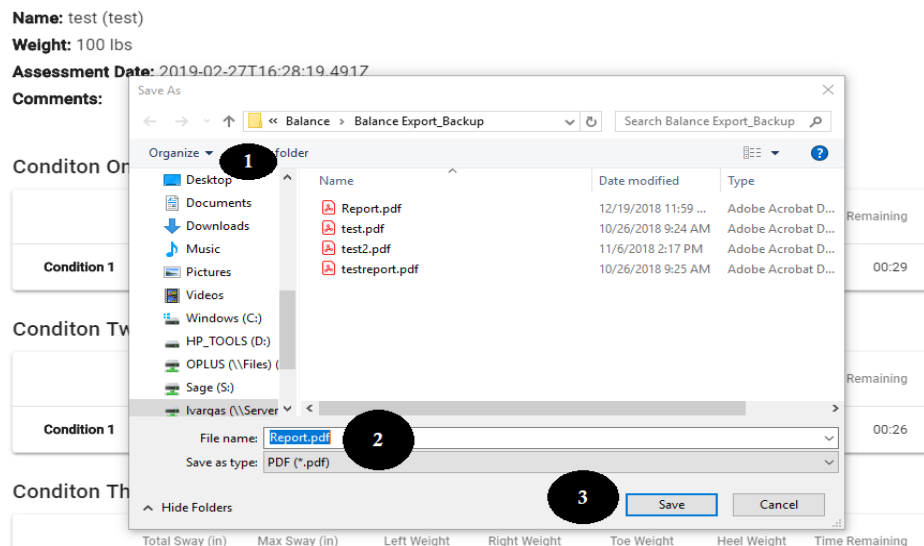

- **After clicking “OK” on Mac**
 1. Select location to save PDF file to
 2. Give the PDF file a name
 3. Click on “Save”

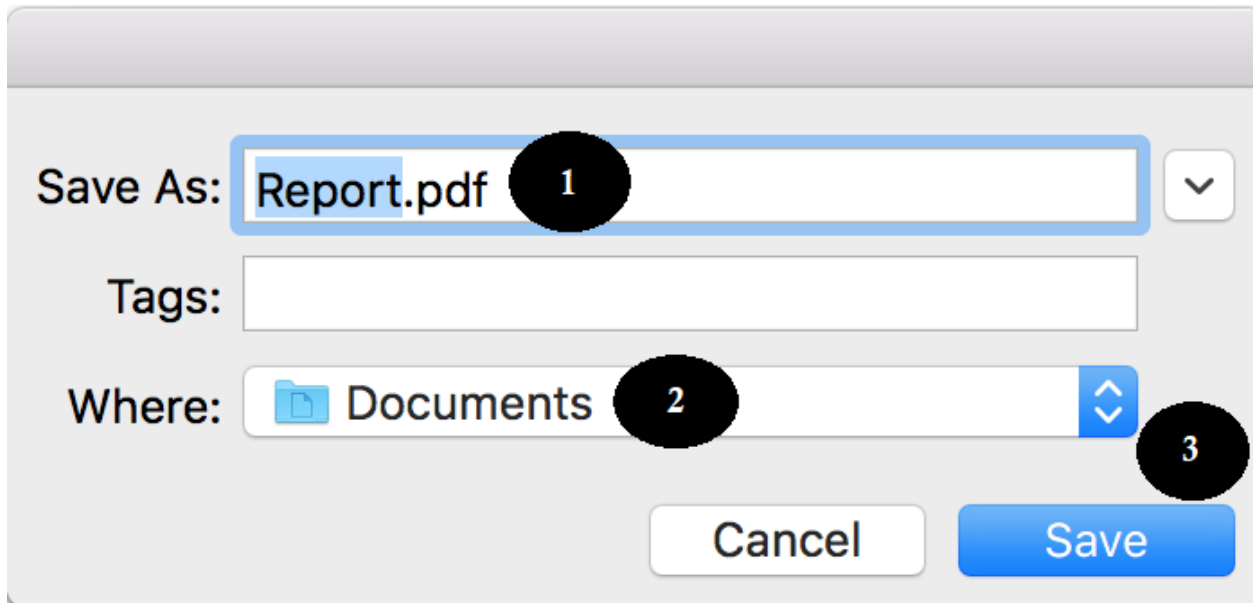

- **After clicking/tapping “OK” on iOS**
 1. Tap on “Save to Files”
 2. Select the location to save PDF file to
 3. Tap on “Add”

